

# BEYOND A BETTER BOX™

XW™-100 AUTOMATED HEMATOLOGY ANALYZER
CLIA WAIVED
SHUTDOWN QUICK GUIDE



# XW-100 SHUTDOWN

### Shutdown

- 1. Press the [➡] button in the upper right corner of the Main Screen; a second screen displays.
- 2. Press the [Move Instrument] button; this will cause the reagents to drain and the XW-100 to shut down.
- 3. Follow the on-screen instructions; you may be prompted to press the Power switch on the right side of the analyzer to complete Shutdown.

## **NOTE:** Once shutdown has been performed:

- 1. The internet connection will need to be reestablished.
- 2. New reagents will need to be scanned to replace existing reagents.

# **NOTES**



Sysmex America, Inc. 577 Aptakisic Road Lincolnshire, IL 60069, U.S.A. Tel. +1 (800) 379-7639 www.sysmex.com/us Sysmex Corporation 1-5-1 Wakinohama-Kaigandori Chuo-ku, Kobe 651-0073, Japan